

# FACT SHEET FOR HEALTHCARE PERSONNEL

Lumin LM 3000 Decontamination UV System for Decontaminating Compatible N95

April X, 2020

Coronavirus  
Disease 2019  
(COVID-19)

You have been given a **N95 respirator or N95-equivalent respirator** (“compatible N95 respirator”) that has been decontaminated using a decontamination system **for single-user reuse by healthcare personnel in a healthcare setting** to help prevent healthcare personnel exposure to pathogenic biologic airborne particulates during the COVID-19 pandemic.

This Fact Sheet informs you of the significant known and potential risks and benefits of the emergency use of decontaminated compatible N95 respirators. These compatible N95 Respirators have been decontaminated using 3B Medical Inc.’s *Lumin LM 3000 Decontamination UV System*, (hereafter also referred to as “**decontaminated N95 respirators**” and or “**Lumin Decontamination Cycle**” throughout this Fact Sheet).

Decontaminated N95 respirators that have been decontaminated using the Lumin LM 3000 Decontamination UV System are authorized for single-user reuse by healthcare personnel in a healthcare setting during the COVID-19 pandemic.

---

**Whether or not you use a surgical mask, respirator, or face shield, always follow infection control measures: wash hands, cover coughs and sneezes, stay home if you may be sick.**

---

## What are the symptoms of COVID-19?

Many patients with confirmed COVID-19 have developed fever and/or symptoms of acute respiratory illness (e.g., cough, difficulty breathing). However, limited information is currently available to characterize the full spectrum of clinical illness associated with COVID-19. Based on what is known about the virus that causes COVID-19, signs and symptoms may appear any time from 2 to 14 days after exposure to the virus. Based on preliminary data, the median incubation period is approximately 5 days, but may range 2-14 days.

Public health officials have identified cases of COVID-19

infection throughout the world, including the United States, which may pose risks for public health. Please check the CDC webpage for the most up to date information.

## What do I need to know about the emergency use of decontaminated N95 respirators?

- The Lumin LM 3000 Decontamination UV System has been authorized for emergency use to decontaminate compatible N95 respirators for single-user reuse by healthcare personnel during the COVID-19 pandemic to prevent wearer exposure to pathogenic airborne particulates.
  - ✓ Compatible N95 respirators are those that are not soft-bodied respirators, foldable respirators, filters made of hydrophilic materials, and respirators with expiratory valves
- The Lumin LM3000 Decontamination UV System meets the performance standards outlined in 21 CFR 880.6600 for Ultraviolet Disinfecting Devices. Testing was submitted to FDA that included UV-C dose measurements and UV irradiance testing for material compatibility on **decontaminated N95 respirators with Lumin LM 3000 Decontamination UV System. The Lumin LM 3000 Decontamination System** demonstrated acceptable performance through 10 decontamination cycles. Note that the Lumin LM 3000 UV System has demonstrated viricidal capability, however it is not sporicidal.

- Decontaminated compatible N95 Respirators are not Sterile.
- The Lumin LM3000 Decontamination UV System inactivates the SARS-CoV-2 virus on compatible N95 respirators
- Testing has not been conducted to show its effectiveness to reduce levels of other pathogens on compatible N95 respirators.

## Preparing compatible N95 respirators for Decontamination

- Place compatible N95 respirators at the end of use into poly bags

**Report Adverse events**, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

# FACT SHEET FOR HEALTHCARE PERSONNEL

Lumin LM 3000 Decontamination UV System for Decontaminating Compatible N95

April X, 2020

Coronavirus  
Disease 2019  
(COVID-19)

- Write name and/or other identifier using a permanent marker so the respirator may be returned after successful decontamination
- Place a tick mark on respirator each time a respirator is prepared for decontamination
- Seal the N95 respirator in the poly bag and place it into area for subsequent decontamination per your healthcare facility's procedures
- **Discard if decontaminated 10 times** or if visibly soiled or damaged

## Use of Lumin LM3000 Decontamination UV System

- Visually inspect the respirator for obvious signs of wear and tear. Discard any respirator with any fluid or stain.
- Use gloves or a disinfecting wipe to open the Lumin drawer to avoid contaminating the outside of the device.
- Place the N95 respirator concave side up on the acrylic rack.
- To ensure decontamination of the straps, remove any tangles or knots from straps so that there is clear UVC light access to the full length of the straps.
- Press the start button. The Lumin LM 3000 System is pre-programmed for a 5-minute cycle.
- Wash hands thoroughly for 20 seconds prior to removing respirator from the Lumin LM 3000 System to avoid cross contamination.
- Each successive decontamination of the respirator should be marked as a check on the respirator
- Discard N95 respirator after 10 uses

**Monitor healthcare personnel for signs and symptoms** of potential infection with SARS-CoV-2 or other respiratory infection for up to and including 14 days after last contact with the SARS-CoV-2 virus and related material, and promptly report such information to 3B Medical Inc.

**Report damage or discoloration** observed upon receipt of the decontaminated N95 respirator, and potential exposure of healthcare personnel from breaks or other damage or degradation of the decontaminated N95

respirators.

Use appropriate personal protective equipment (PPE) when caring for individuals suspected of having COVID-19 as outlined in the CDC webpages, including *Interim Infection Prevention and Control Recommendations for Patients with Confirmed Coronavirus Disease 2019 (COVID-19) or Persons Under Investigation for COVID-19 in Healthcare Settings, Infection Control, and FAQ about PPE*.

Current information on COVID-19 for healthcare personnel is available at CDC's webpage, *Information for Healthcare Professionals* (see links provided in "Where can I go for updates and more information" section).

## What are the known and potential benefits and risks of using decontaminated N95 Respirators?

Potential benefits include:

- May help prevent exposure to airborne pathogens, and therefore reduce the risk of infection or illness
- Extends the usability of compatible N95 or N95-equivalent respirators by allowing for decontamination and single-user reuse.

Potential risks include:

- Inadequate disinfection due to improper placement of multiple items or masks during a single cycle. UVC systems work optically and are ineffective due to shadowing of other objects.
- Failure of filtration efficiency
- Reduced breathability
- Strap failure and ineffective face-fit
- Reused respirators may not have been effectively decontaminated of SARS-CoV-2 or other pathogens

## Overview of the Lumin LM 3000 Decontamination UV System

The Lumin LM 3000 Decontamination UV System is a self-contained decontamination device that is normally used as an adjunct for cleaning CPAP Accessories (Masks, Hoses, and Water chamber). For this emergency use, the Lumin LM 3000 Decontamination UV System is intended for decontamination of compatible N95 or N95-equivalent

**Report Adverse events**, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

# FACT SHEET FOR HEALTHCARE PERSONNEL

Lumin LM 3000 Decontamination UV System for Decontaminating Compatible N95

April X, 2020

Coronavirus  
Disease 2019  
(COVID-19)

respirators that are contaminated or potentially contaminated with SARS-CoV-2 and other pathogenic microorganisms (note: this treatment is not sporicidal). **N95 respirators that are soft-bodied respirators, foldable respirators, filters made of hydrophilic materials, and respirators with expiratory valves are not compatible with the Lumin LM 3000 UV decontamination process**

The Lumin LM3000 Decontamination UV System works by delivering a relatively high dose of ultraviolet germicidal irradiation (UVGI). Each decontamination cycle results in an estimated irradiance of 60 mJ/cm<sup>2</sup> in a 60 second cycle, or 300 mJ/cm<sup>2</sup> in a 5 minute cycle. The Lumin LM3000 Decontamination UV System can decontaminate **one (1)** mask per 5 minute cycle. The Lumin LM3000 Decontamination UV System enables single-user reuse of compatible N95 respirators that would otherwise be disposed of after a single use. However, respirators that are visibly soiled must be discarded and not reused or decontaminated.

## What is an EUA?

The United States FDA has made the emergency use of the Lumin LM 3000 Decontamination UV System to decontaminate compatible N95 respirators available under an emergency access mechanism called an Emergency Use Authorization (EUA). The EUA is supported by the Secretary of Health and Human Service's (HHS's) declaration that circumstances exist to justify the emergency use of medical devices, including alternative products used as medical devices, due to insufficient supply during the COVID-19 pandemic.

The Lumin LM 3000 Decontamination UV System has been made available under an EUA, and has not undergone the same type of review as an FDA-approved or cleared device. FDA may issue an EUA when certain criteria are met, which includes that there are no adequate, approved, available alternatives, and based on the totality of scientific evidence available, it is reasonable to believe that the Lumin LM 3000 Decontamination UV System may be effective at preventing healthcare personnel exposure to pathogenic airborne particulates during periods of insufficient respirator supply during the COVID-19

pandemic by decontaminating compatible N95 respirators, for a maximum of 10 decontamination cycles per respirator, that are contaminated or potentially contaminated with SARS-CoV-2 or other pathogenic microorganisms. Note that this treatment is not sporicidal.

The EUA for the Lumin LM 3000 Decontamination UV System by 3B Medical is in effect for the duration of the COVID-19 declaration justifying emergency use of medical devices, unless terminated or revoked (after which the products may no longer be used).

---

## Where can I go for updates and more information?

### **CDC webpages:**

**General:** <https://www.cdc.gov/COVID19>

### **Healthcare Professionals:**

<https://www.cdc.gov/coronavirus/2019-nCoV/guidance-hcp.html>

**Infection Prevention and Control Recommendations in Healthcare Settings:** <https://www.cdc.gov/coronavirus/2019-ncov/infection-control/control-recommendations.html>

**Infection Control:** <https://www.cdc.gov/coronavirus/2019-ncov/infection-control/index.html>

### **FAQ on Personal Protective Equipment:**

<https://www.cdc.gov/coronavirus/2019-ncov/hcp/respirator-use-faq.html>

### **FDA webpages:**

**General:** [www.fda.gov/novelcoronavirus](http://www.fda.gov/novelcoronavirus)

**EUAs:** <https://www.fda.gov/medical-devices/emergency-situations-medical-devices/emergency-use-authorizations>

---

**Report Adverse events**, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**